

END OVERDOSE.

1

FILL A SMALL CONTAINER WITH AT LEAST 15 ML OF WATER (SEE DILUTION GUIDE FOR MOST ACCURATE RESULTS)

2

FULLY DISSOLVE A SMALL AMOUNT OF POWDER (I.E. RESIDUE ON BAG / SCRAPED & CRUSHED PILL-NOTE EACH PILL MUST BE TESTED INDIVIDUALLY)

WHAT DOES 15 ML LOOK LIKE?



WHAT CONTAINERS CAN YOU USE?





3

HOLD THE **BLUE** END OF THE STRIP AND DIP IT THE WATER FOR ABOUT **10 SECONDS** (UNTIL WATER IS IN THE WHITE PART OF THE STRIP)

4


TAKE THE STRIP OUT OF THE WATER AND WAIT **60 SECONDS**

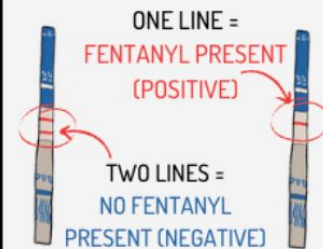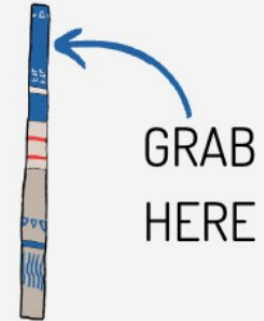

HOW TO USE FENTANYL TESTING STRIPS

METH OR MDMA

1 tsp of water 

10 mg of powder 

ALL OTHER DRUGS

1 tsp of water 

50 mg of powder 